

ANTIOXIDANT PROTOCOL

Skintrinsiq treatments provide customized experiences to refresh and revitalize the appearance of your skin.

The Skintrinsiq Antioxidant Treatment is intended for clients with visible damage who are interested in taking the first step to fortify the skin's natural defenses against future environmental assaults. This antioxidant treatment is perfect for those with photodamage.

30
MINUTES*

SUPPLIES

- Disposable headband
- Gloves (as needed)
- Esthetic wipes
- Cotton rounds

REQUIRED PRODUCTS

- Obagi Nu-Derm® Gentle Cleanser
- Obagi Nu-Derm® Toner
- Blue Peel Radiance® (Optional)
- Professional-C® Serum 10%, 15%, or 20%
- Professional-C® Peptide Complex
- Obagi Sun Shield™ Matte Broad Spectrum SPF 50 Sunscreen

CONSIDERATIONS

Recommended wait times before a Skintrinsiq treatment:

2 DAYS	Tretinoin, salicylic acid, or benzoyl peroxide treatment
5 DAYS	Facial waxing, laser hair removal, or electrolysis
7 DAYS	Injectables or dermal fillers
14 DAYS	Chemical peel, microneedling, microblading
4 WEEKS	IPL/Photofacial
3 MONTHS	Laser treatment/cosmetic surgery
6 MONTHS	Accutane, chemotherapy, radiation

*45 minutes if the Blue Peel Radiance Option is added.

STEP	PRODUCT/DEVICE	AMOUNT	ACTION
CONSULTATION			
1. Cleanse	Obagi Nu-Derm Gentle Cleanser	Dime-sized amount per cleanse	Double cleanse face; remove cleanser with esthetic wipes
2. Prep + Tone	Obagi Nu-Derm Toner	Nickel-sized amount	Apply with 1 cotton round; sweep face, neck and décolleté
3. Skin Analysis	Mag Lamp	N/A	2 cotton rounds to cover eyes
4. Skintrinsiq Extraction	Skintrinsiq Device	N/A	Use device as directed on intended treatment area
4a. Advanced Option	Blue Peel Radiance	½ or 1 vial	Apply as directed
4b. Cleanse	Obagi Nu-Derm Gentle Cleanser	Dime-sized amount	Work into entire face and remove with tepid, wet sponges
5. Clarify	Obagi Nu-Derm Toner	Nickel-sized amount	Use as needed to remove surface debris
6. Skintrinsiq Infusion	Skintrinsiq Device and Professional-C Serum 10%, 15%, or 20%	8-10 drops	Apply product to face and use device as directed on intended treatment area
7. Moisturize	Professional-C Peptide Complex	1 pump	Apply with hands to face, neck and décolleté
8. Protect	Obagi Sun Shield Matte Broad Spectrum SPF 50 Sunscreen	½ tsp	Apply with hands to face, neck and décolleté
Recommend an at-home Obagi regimen so patients can see the prolonged benefits of today's Skintrinsiq treatment.			



All products/brand names, whether designated by notice or not (®/™), are trademarks of Obagi Cosmeceuticals LLC and/or its affiliates. Distributed by Obagi Cosmeceuticals LLC. 2022 Obagi Cosmeceuticals LLC. All rights reserved. 02/22 851103-2

OBAGI®