

CLARIFYING PROTOCOL

Skintrinsiq treatments provide customized experiences to refresh and revitalize the appearance of your skin.

The **Skintrinsiq Clarifying Treatment** is intended for clients who have oily skin that is prone to breakouts.

30
MINUTES

SUPPLIES

- Disposable headband
- Gloves (as needed)
- Esthetic wipes
- Cotton rounds

REQUIRED PRODUCTS

- Obagi Nu-Derm® Gentle Cleanser
- Obagi Nu-Derm® Toner
- CLENZIderm® M.D. Pore Therapy
- CLENZIderm® M.D. Therapeutic Lotion (Optional)
- Obagi360™ Retinol 0.5
- CLENZIderm® M.D. Therapeutic Moisturizer
- Obagi Sun Shield™ Mineral Broad Spectrum SPF 50 Sunscreen

CONSIDERATIONS

Recommended wait times before a Skintrinsiq treatment:

2 DAYS	Tretinoin, salicylic acid, or benzoyl peroxide treatment
5 DAYS	Facial waxing, laser hair removal, or electrolysis
7 DAYS	Injectables or dermal fillers
14 DAYS	Chemical peel, microneedling, microblading
4 WEEKS	IPL/Photofacial
3 MONTHS	Laser treatment/cosmetic surgery
6 MONTHS	Accutane, chemotherapy, radiation

STEP	PRODUCT/DEVICE	AMOUNT	ACTION
CONSULTATION: Not Recommended for Sensitive Skin			
1. Cleanse	Obagi Nu-Derm Gentle Cleanser	Dime-sized amount per cleanse	Double cleanse face; remove cleanser with esthetic wipes
2. Prep + Tone	Obagi Nu-Derm Toner	Nickel-sized amount	Apply with 1 cotton round; sweep face, neck and décolleté
3. Skin Analysis	Mag Lamp	N/A	2 cotton rounds to cover eyes
4. Skintrinsiq Extraction	Skintrinsiq Device	N/A	Use device as directed on intended treatment area
5. Clarify	CLENZIderm M.D. Pore Therapy	4-5 pumps	Apply with 1 cotton round; allow to dry for 5 minutes
5a. Advanced Option	CLENZIderm M.D. Therapeutic Lotion	2 pumps	Apply with hands to face ; allow to dry for 5 minutes
6. Skintrinsiq Infusion	Skintrinsiq Device and Obagi Retinol 0.5%	½ gram	Apply product to face and use device as directed on intended treatment area
7. Moisturize	CLENZIderm M.D. Therapeutic Moisturizer	1-2 pumps	Apply with hands to face, neck and décolleté
8. Protect	Obagi Sun Shield Mineral Broad Spectrum SPF 50 Sunscreen	½ tsp	Apply with hands to face, neck and décolleté

Recommend an at-home Obagi regimen so patients can see the prolonged benefits of today's Skintrinsiq treatment.



All products/brand names, whether designated by notice or not (®/™), are trademarks of Obagi Cosmeceuticals LLC and/or its affiliates. Distributed by Obagi Cosmeceuticals LLC. 2022 Obagi Cosmeceuticals LLC. All rights reserved. 02/22 853102-2

